Autoclave para esterilización 1.5 Litros
1.5 liter sterilization autoclave

AUTESTER P 1.5

4001726

1 Seguridad

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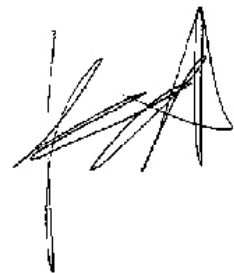
1 Safety

Safety use of autoclaves.

In order to be respected, across this manual, the risk of danger situations are remarked.

1.1 Safety symbols

It identify the risk and the safety actions to be applied.

The symbols make reference to the paragraph marked with a grey line.

Hazard of electrical shock

Electric hazard for entering in areas marked with this signal, or for following the manual instruction signalled by this icon.

Follow the instructions for the describe operation.

Hazard of burns to be in contact with hot surfaces.

Temperature in this area can exceed 60°C. Use heat protected gloves for the described action.

Follow the instructions for the describe operation.

Important information

- Information important to obtain good results or for an optimal operation of the equipment.
- Information important to extend the life of the equipment or to avoid the degradation of some of its components.

1.2 Risks which the operator is put under

Autoclave uses exposes to the operator to the following situations of risk:

- Possibility of touching parts to a temperature above to 60°C
- Possible be in contact with hot steam.
- Electrical risk.

1.3 Personnel training

This equipment must only operated by personnel trained an suitable qualified to work in a laboratory and its common hazards.

This equipment must only operated by personnel who have read and understood this manual or have been qualified on its manipulation and operation.

1.4 Proper uses of the equipment

This equipment is for indoor laboratory use, for sterilizing loads as described.

The sterilization assurance needs also proper procedures for cleaning and storing the loads.

The improper use of this unit (not following manual instructions) can endanger the protection prepared by the equipment.

This equipment its not designed and not manufactured to work in an explosive ambient area.

1.5 Modifications

The operation modification or the manipulation of the safety systems unauthorized by the manufacturer, could expose the operator to hazards not indicated in this manual.

2 Packing list

Components	Qty
• Basket	1
• Power cord	1
• Fuse	1
• Drain water pipe	1
• Basket holder	1
• Used water tank	1
• Spare for door gasket	1
• Funnel	1

3 Overview

The AUTESTER P 1.5L autoclave is class "N". Suitable to sterilize metallic or autoclavable plastic instruments, without hollow areas and not wrapped. Used for odontologist, veterinary and, general speaking, to sterilize loads that could be in contact with blood, as "tattoo" and piercing centres, beauty centres, hairdresser, ...

As this autoclave does not have drying function, and neither can sterilize wrapped loads, the sterilized load must be used before low time (2-3h) after its sterilization.

The AUTESTER ST DRY PV Class B includes the following features:

- Stainless steel sterilization chamber.
- External cover with epoxy coat.
- Digital control.
- Separated Clean/used water tanks.

To get assure load sterilization the proper program must be used. Follow this instruction manual.

3.1 Cycles

- Fast sterilization at 134°C for thermal-resistant load.
- Sterilization at 121°C for autoclavable load at 121°C.

Sterilization cycles

Phase	Cycle 134°C	Cycle 121°C
Temperature	134	121
Pressure	2.10	1.10
Class	N	N
Sterilization time	3.5 min	6 min
Total time	2+2+3.5 min	2+2+6 min (No warmup)
Use	Metallic	Plastic-rubber

3.2 Guidelines to get test an efficient sterilization

The material to be sterilised needs to be clean, free from any encrusted or residual material, wash it in a good detergent and distilled water, then rinse with distilled water.

Do not overload the trays, left a free space of 1 to 2 cm between goods to allow a free steam circulation and to ease drying.


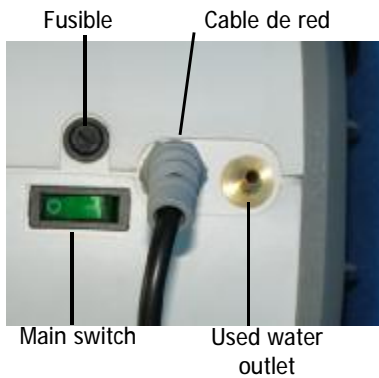
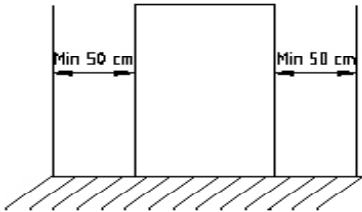
Fig 4.2

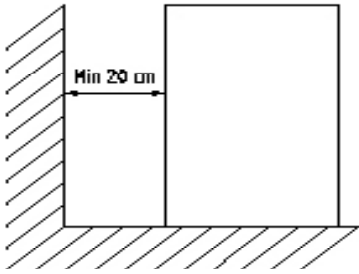Fig 4.3

4 Installation

Once the unit is unpacked, visually check, the packing list (See 1.1) and that there are no parts damaged during transportation.

Save the packing for some days for later sending.

4.1 Suitable location

Place the unit over a flat level and stable surface, suitable for the unit weight.

No longer that 1.5m from a power supply plug.

No tap water and waste will be needed.

To work comfortably, let a free space of 50cm on each side.

To allow ventilation let and not least than 20cm on the rear part.

Do not block or cover any ventilation holes.

Prepare to refill the water reservoir from the top of the unit.

4.2 Connection to power supply

Locate a power supply socket (Rated to 16A) near to the unit.

For safety reason the socket must have ground terminal, since the unit must work grounded.

Do not manipulate the power cord with wet hands.

Before to plug on the socket check that its voltage match the voltage marked on the unit plate.

Plug the power cord on the autoclave rear part.

4.3 Filling water reservoir.

Use distilled or deionized water with this autoclave.

Use the supplied funnel to refill the tank.

Fill the tank with approx. 1.5 litre . See Fig 4.3

4.4 Connection to used water tank

Use the supplied hose to connect the used water tank to autoclave.

Connect the hose from the water outlet to the used water tank inlet. See Fig 4.2.

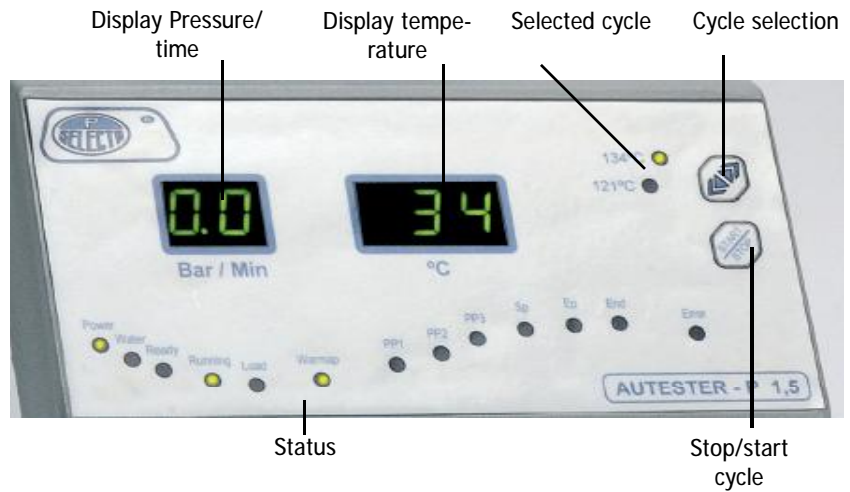
Used water tank

5 Control panel

The control panel allows function selection and shows operation status.

Press main switch to power on the autoclave. See Fig 4.2

- **Display pressure /min:** Pressure (bar) or remaining time.
- **Display temperature:** Temperature (°C) on the chamber
- **Status indication:** Autoclave status indication:

Power:	Unit powered On.
Water:	Water level on the reservoir. Lights ON when water must be added. (Add water)
Ready:	Ready to start. (Press START)
Running:	Cycle running
Load:	Ready to load material to sterilize
Warm-up:	Warm up. (Up to temperature reach 100°C)
PP1, PP2:	Thermal vacuum. (Display indicates pressure)
PP3:	Rise to sterilization temperature. (Display indicates pressure)
SP:	Sterilization. (Display indicates time to finish)
EP:	Draining water
END:	Cycle end. Open can be opened
ERROR:	Error on the operation. See table

Error table.

Item	Error Code	Failure	Action to take
1	E1	Temperature sensor error	Check sensor
2	E2	Over pressure (>2.6bar).	Check pressure sensor
3	E3	Chamber over temperature.	Check temperature sensor
4	E4	Keep warm keep pressure.	
5	E5	Exhaust failed (20s < 0.3bar)	Check valve
6	E6	Door open on working time.	Check door look
7	E7	Steam generator Overtemperature	Check temperature sensor
8	E8	Pre-heating fault >20min	Check cylinder heaters
9	E9	Filling steam gen. fails >30min	Check steam pipes

6 Operation

1. Switch on the autoclave. Main switch.
2. Open the chamber door. Remove the baskets.
3. Place the load into the basket. Insert on the chamber. Close the door and lock. Rotate handler 1/4 turn.
4. Select a cycle. (121°C, 134°). Press Start.
5. If water light is ON, add water on the reservoir.
6. The unit goes along the different phases, until END indication.
7. Open the door to remove the load from the chamber.
8. The unit is ready for another cycle.

Note:

To avoid problems caused by dirty:

- Clean the goods before to sterilize it.
- Specially clean free from oil and grease.

Caution: On unloading material from autoclave.

Hazard of burns to be in contact with hot surfaces.

Temperature in this area can exceed 100°C.

Use the tray holder to load or unload the tray with instruments in it in order to avoid burning injuries.

7 Technical specification

Mains voltage:	230V 50Hz (See plate) Other power supply voltage could be available (Ask to your dealer)
Electrical power:	1100W (See plate)
Maximum consumption:	8.5 A
Fuses:	8A 20x5Ø mm
Weight (Net):	17.2 Kg
Dimensions: W x L x H	200 x 340 x 450 mm
Chamber volume:	1.5 litre
Chamber dimensions	Ø 90mm x 160mm
Water reservoir volume:	1.5 litres
Ambient conditions:	Inner use. Altitude up to 2000m Temperature from 5°C to 40°C. Relative maximum damp 80%for above 31°C temperatures, decreasing lineally to 50% for 40°C.
Overvoltage category:	Category II
Pollution degree:	2
Door and front material:	ABS injected
Carriage material:	Aluminium

7 Maintenance and service

To get a good performance from your autoclave read this instructions.

Fig. 7.1 Junta del cierre

7.1 Lock gasket maintenance

Avoid to damage the lock gasket with the load.

Keep clean the gasket from particles

Clean periodically with non abrasive cloth

If steam leakage is observed, change it.

Fig. 7.2 Empty reservoir

7.2 Clean the water reservoir

- The hose for empty the water reservoir it under the autoclave. Fig 7.2
- Pop from the hose stopper.
- Open the stopper and collect the water on a suitable flask

Fuse

Fig. 7.4 Fuse changing

7.3 Fuse changing

Follow the instruction:

- Unplug the unit from mains.
- Use a screwdriver to rotate a 1/4 turn the fuse holder.
- Use fuse 5x20mm 8A
- Insert the fuse and close the fuse holder cover.

7.4 Other service issues

Ask your dealer for qualified J.P. SELECTA,s,a. service

**ATTENTION
DANGER
OF ELECTRICAL HAZARD**

Unplug the unit from the mains to made any operation inside the autoclave.

10 Warranty

The warranty covers the defective components or faulty assembling of the unit for 1 year since the purchasing date.

Keep purchasing bill as the only valid document for the purchasing date.

The warranty does not covers the damages caused for a unit malfunction. The unit must be suitably qualified before to be used for valuable samples processing.

Return good are not accepted without previous J.P. SELECTA, s.a. approval.

11 Statement of conformity CE

The manufacturer:

J.P. SELECTA, s.a.
Ctra. NII Km 585.1
08760 ABRERA (BARCELONA)
ESPAÑA

Declares that the laboratory equipment:

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Meet the following European Directives:

73/23/CEE	Directive for electrical safety.
89/336/CEE	Directive for electromagnetic compatibility.(EMC)
97/23/CEE	Directive for pressure devices. Paragraph 3.3

Meet the following Standarts:

EN 61326:2002
Electrical equipment for measurement, control and laboratory use. EMC Requirements.

EN 61010-1:2002
Safety requirements for electrical equipment for measurement, control and laboratory use. Part 1: General requirements.

EN61010-2-041
Particular requeriments for laboratory autoclaves.

Abreira, June 2007

Ramón Ramón
Dtor. Técnico.

David Pecanins
Rble. Calidad

J.P. SELECTA, s.a.