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**Sterilization Autoclave Class B 18L / 23L
with a build-in printer**

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
Index

1 Safety	12
1.1 Safety icons	12
1.2 Risks to which the operator is subjected.....	12
1.3 Personnel qualification	12
1.4 Equipment use	12
1.5 Modifications	12
2 Packing list & accessories	13
3 Description	13
3.1 Autoclaves type «B»	13
3.1 Preset programs.....	14
4 Installation	14
4.1 Location.....	14
4.2 Connection to the mains	14
4.3 Filling the water tank	15
4.4 Draining.....	15
5 Control panel	15
6 Operation.....	16
6.1 Program selection.....	16
6.2 Sterilization	16
6.3 Configuration	17
7 Sterilization Record Reading Software	17

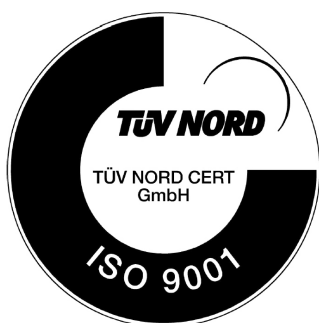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1 Safety

Safety instruction for the use of autoclaves.

Risky situations that have to be respected are indicated along this manual.

1.1 Safety icons

Identify risky situations and safety measurements that can be taken. Icons refer to the paragraph marked with a grey line.

Electrical hazard

Risk of electrical accidents when accessing the areas marked with this sign or when performing the operations described in this manual by this icon.

Respect the indicated instructions to perform the operation.

Risk of burns due to high temperature areas

The temperature in the area indicated by this icon can exceed 60°C. Use anti thermal gloves to perform the operation described.

Respect the indicated instructions to perform the operation.

Important information

- Important information to achieve good results or for optimal operation.
- Important information to extend the equipment life or prevent degradation of any component.

1.2 Risks to which the operator is subjected

- Possibility of touching parts at a temperature higher than 60°C.
- Possible exposure to vapours.
- Electrical hazard.

1.3 Personnel qualification

This equipment may only be used by personnel who have been properly trained.

This equipment may only be used by personnel who have read and understood these instructions or has been sufficiently qualified in the operation of this equipment.

1.4 Equipment use

This equipment is intended for use in laboratories (interior) for sterilizing materials.

Using the equipment in a way other than the indicated in this manual could compromise the protection provided by the equipment.

It cannot be used with explosive or flammable substances or to work in explosive atmospheres.

1.5 Modifications

The modification of the operation or the equipment safety systems handling are not authorized by the manufacturer, as it may expose the operator to risks not specified in this manual.

2 Packing list & accessories

The standard equipment consists of the following components:

Components	Units
• Ventilated shelf	3
• Electrical mains connection cable	1
• Water drainage tube	1
• Handle for handling trays	1
• Door gasket (spare part)	1
• Water filling hose	1
• Tray support	1

3 Description

Autoclaves Class B are widely used in medical, dental, veterinary and generally to sterilize materials that may come in contact with blood, as for instance, tattoo centres, piercing salons, hairdressers or beauty centres, etc.

Autoclaves Class B assure proper sterilization of different types of normal processes: solid, solid packed, wrapped porous and hollow. To ensure sterilization, Autester ST DRY PV autoclaves achieve a complete air removal and a better steam penetration thanks to an efficient system of fractionated vacuum.

AUTESTER ST DRY PV Class B print autoclaves have the following features:

- Sterilization chamber and cover in stainless steel.
- External epoxy coated case.
- External steam generator to the sterilization chamber.
- Operations digital control.
- Specific programs for testing and autoclave diagnosis.
- Large backlight LCD display.
- Air inlet filter to the camera.
- Water tank.
- RS-232 output for printer and USB connection for logs downloading.
- Build-in printer.

To guarantee sterilization of the load, the appropriate program indicated in the instructions must be used.

3.1 Autoclaves type «B»

Autester ST DRY PV B autoclaves allow sterilization of:

- Solid load resistant to temperature: instrumental, metallic.
- Solid load sensitive to temperature: Plastics
- Solid packed load.
- Porous load: textiles
- Wrapped porous load.
- Catheters, tubes and cavities devices.

This autoclave is not suitable for sterilizing liquids.

3.2 Preset programs

Programs (Cycles)

Program	1	2	3	4	5	6
Program type	Emergency	Solid	Hollow	Sanguinary	Solid	Hollow
Sterilization temp	134°C	134°C	134°C	121°C	121°C	121°C
Sterilization pressure	2.10 bar	2.10 bar	2.10 bar	1.10 bar	1.10 bar	1.10 bar
Sterilization	4 min	4 min	4 min	18 min	20 min	20 min
Drying	4 min	8 min	5 min	18 min	8 min	20 min

Program	7	8	9	10	11	12
Program type	Glass	Tampon	Test B&D	Test Helix	User	Test vacuum
Sterilization temp	134°C	121°C	134°C	134°C	Free	--
Sterilization pressure	2.10 bar	1.10 bar	2.10 bar	2.10 bar	1.1/2.1 bar	--
Sterilization	4 min	30 min	3.5 min	3 min	Free	--
Drying	4 min	18 min	3 min	3 min	Free	--

4 Installation

Once the equipment is unpacked, visually check if it's in good conditions. Keep the original packaging for a few days.

4.1 Location

Place the machine on a flat, levelled stable surface, suitable for the equipment weight.

Use the supplied leveller to level the autoclave.

Note: The autoclave should be slightly raised from the front side to ensure the water in the boiler not to accumulate back in the drain area. Otherwise, the "drying" phase will be incorrect and may cause the error message E3.

You will need a plug, no longer than 1m, in accordance with the equipment features.

There's no need of a water tap nor a drain.

To work comfortably, leave free about 50cm around the equipment.

To ensure air circulation, leave free about 20cm behind the equipment.

Do not block any ventilation grid.

4.2 Connection to the mains

Locate an outlet (type 16A) near the autoclave.



Use the network connection cable supplied or a similar one. (Section 1.5mm²)

The plug must be grounded.

Do not handle power cords with wet hands.

Check that the plug voltage matches with the indicated on the features label.

Connect the cable to the back side of the autoclave.

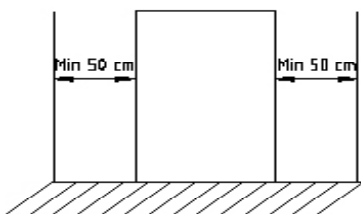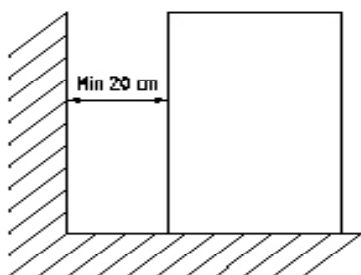4.3 Filling the water tank

Use deionized water for the autoclave. Fill the tank with 5 litres approximately through the top hole (3.5L for the 18L autoclave and 4L for the 23L one).

Another filling method is by means of the water inlet 9. For this, connect the provided hose, immerse the other end into a container with fresh water. Press key 2 for 3 seconds to start the filling process. The equipment stops the process when it is completely filled.

4.4 Draining

The autoclave has two separate tanks, one for clean water and the other for used water.

Both tanks are separately emptied, through the valves on the right front side.

Connect a hose to open the valve.


5 Control panel

- 1 Display
- 2 Increase value key
- 3 Decrease value key
- 4 Program key
- 5 Start/Stop key

- 6 Main switch.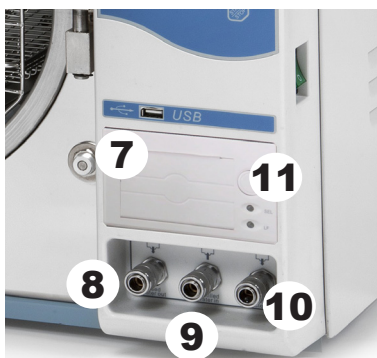

- 7 USB port
- 8 Used water draining connector
- 9 Optional automatic filling connector
- 10 Clean water draining connector
- 11 Printer

PROGRAM CONFIGURATION

6 Operation

6.1 Program selection

- Turn on main switch 6.
- Press PROG 4 key to access the options: PROGRAM and CONFIGURATION.
- Press the 2 Up key to access the preset programs (page 4).
- Select the program with keys 2 and 3.
- Confirm by pressing PROG 4 key.


6.2 Sterilization

- Load the trays with the material to be sterilized.
- Press the 5 Start/Stop key to start the selected program.


If this operation is made with the door closed, the display will ask to open the door and close again before pressing the Star/Stop key.

- The display will indicate the process status every moment.
- At the end, END will appear. Wait for the 0 bar condition.

Process Display: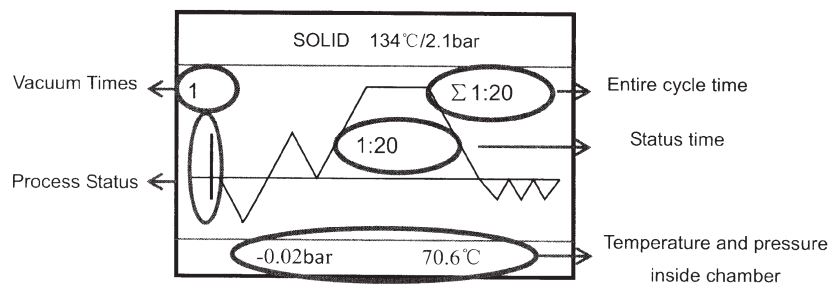

Graphic of process shown on the display.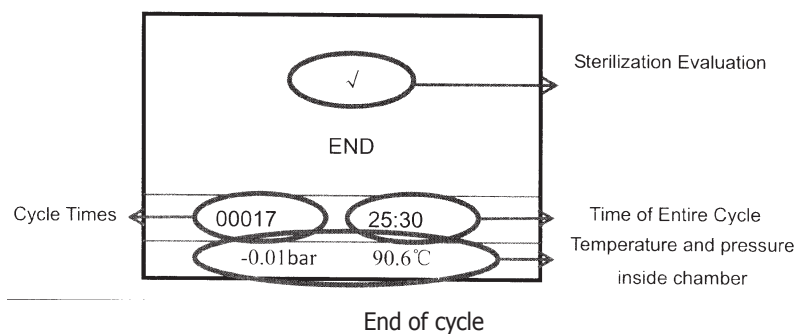


End of cycle

6.2.1 Setup program 11 USER

This special program is used to achieve a perfect sterilization. The process to setup it is:

- Press the PROG key (4) to enter program 11, then press PROG (4) to select the value you want to change: sterilization temperature, sterilization time, drying time and vacuum times.
- Press the arrow keys (2) or (3) to vary the value of each parameter.
- Press the START key (5) to confirm the modification.
- Press START (5) again to start the cycle.

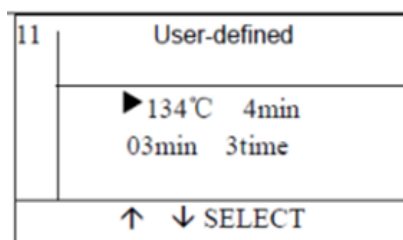Sterilization temperature	121°C	134°C
Sterilization time	15 a 50 minutes	4 to 40 minutes
Drying time	1 a 30 minutes	1 to 30 minutes
Vacuum times	0, 1 or 3 times	0, 1 or 3 times

TIME
PRINTER
LANGUAGE

6.3 Configuration

- Turn on the main switch (6).
- Press the Up Key (3) to access the configurable parameters (Time, Printer, Language).
- Select the option to modify with keys 2 and 3.
- Press the 5 Start/Stop key to access the submenu or choose the desired option

Caution when removing the material from the autoclave

Risk of burns due to contact with hot surfaces.

The material inside the autoclave could be at a temperature higher than 100°C.


Use the handle provided for trays handling.

7 Sterilization Record Reading Software

1. Double click the desktop shortcut (after the program is installed)


2. The reading software operation screen will appear.

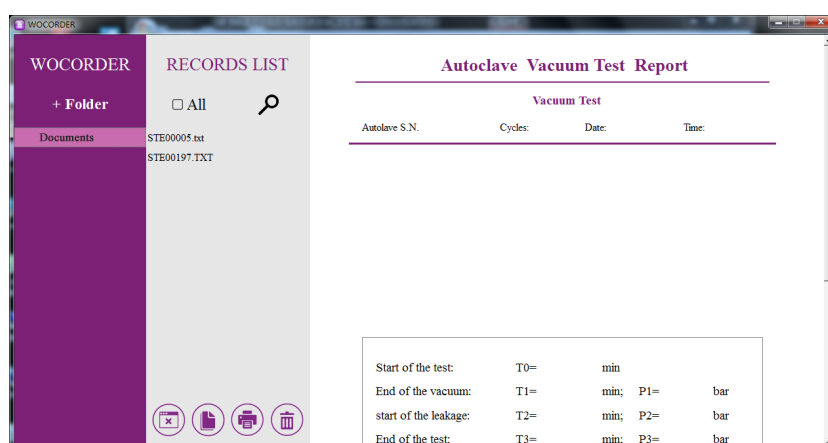
3. Click the left mouse button.


4. The following dialog box will appear: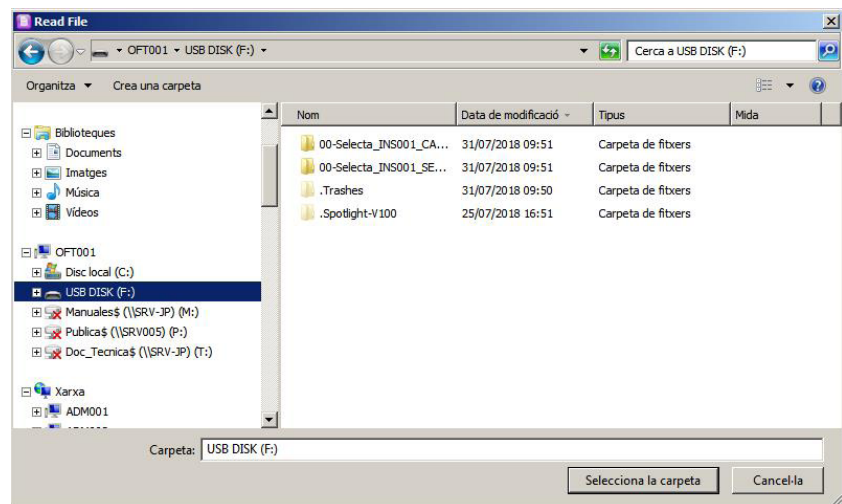


5. Click the **"Choose this folder"** button, then return to the Reading Software main screen. Or click the **" + Folder "** again, and you can continue choosing another folders.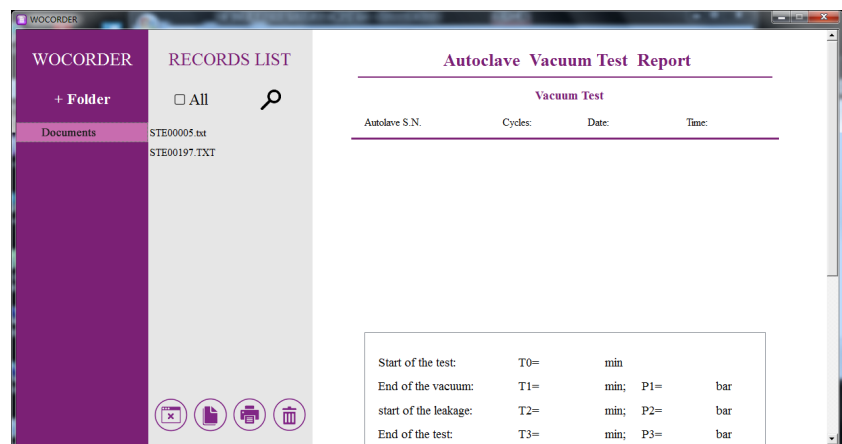


6. Choose one folder in the left column, then it will show you the records in the middle column. If you choose another folder at the same time, they also can be shown in the middle column.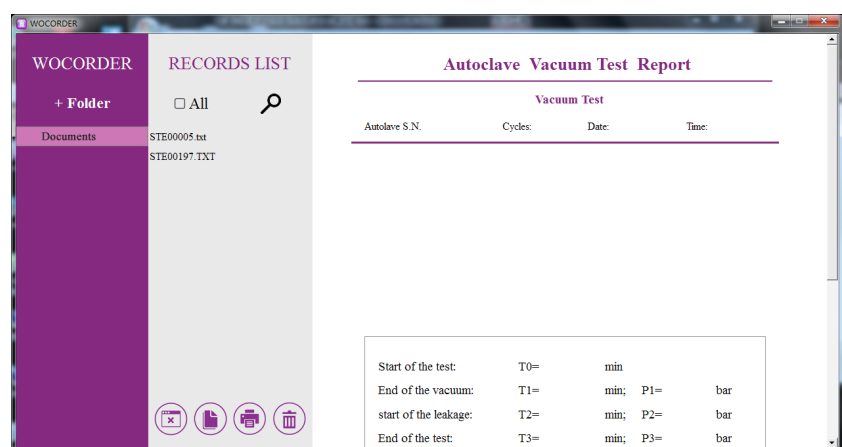


7. When you choose one of these records in the middle column, the log details will be shown in the right screen. If you choose all of the records, it will only show you the first one.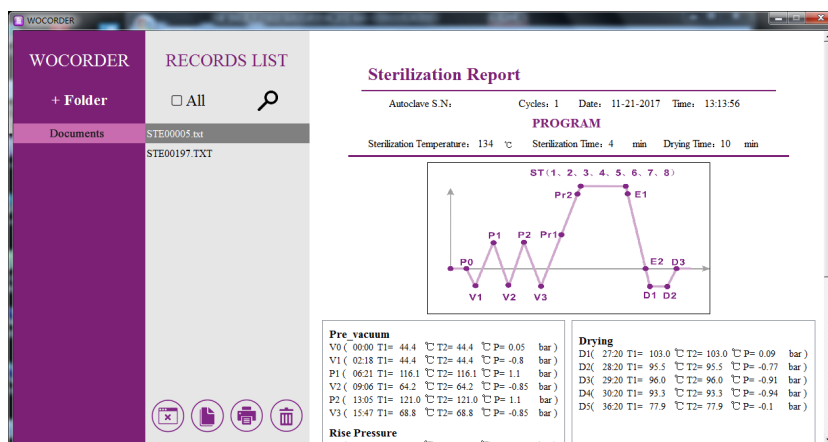

8. Press this button, a dialog box will show up to confirm whether you want to delete all these records in the middle column: choose "yes" and delete all; or choose "no" to return.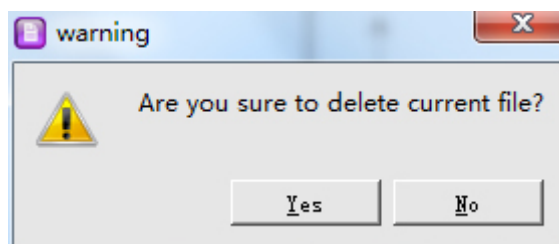

9. Press this button in the middle screen, and copy the chosen records to any document you want.

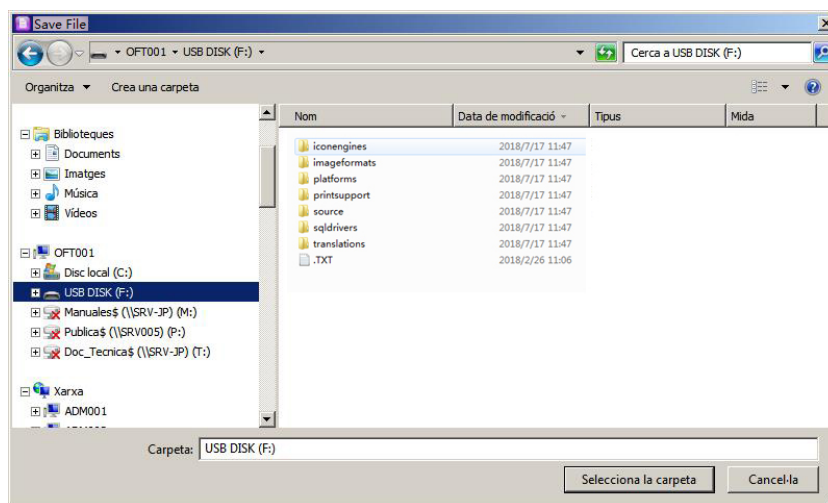


10. Press this button, print the chosen record, a print preview screen will

jump out. Press on the upper right corner, then you can print the record.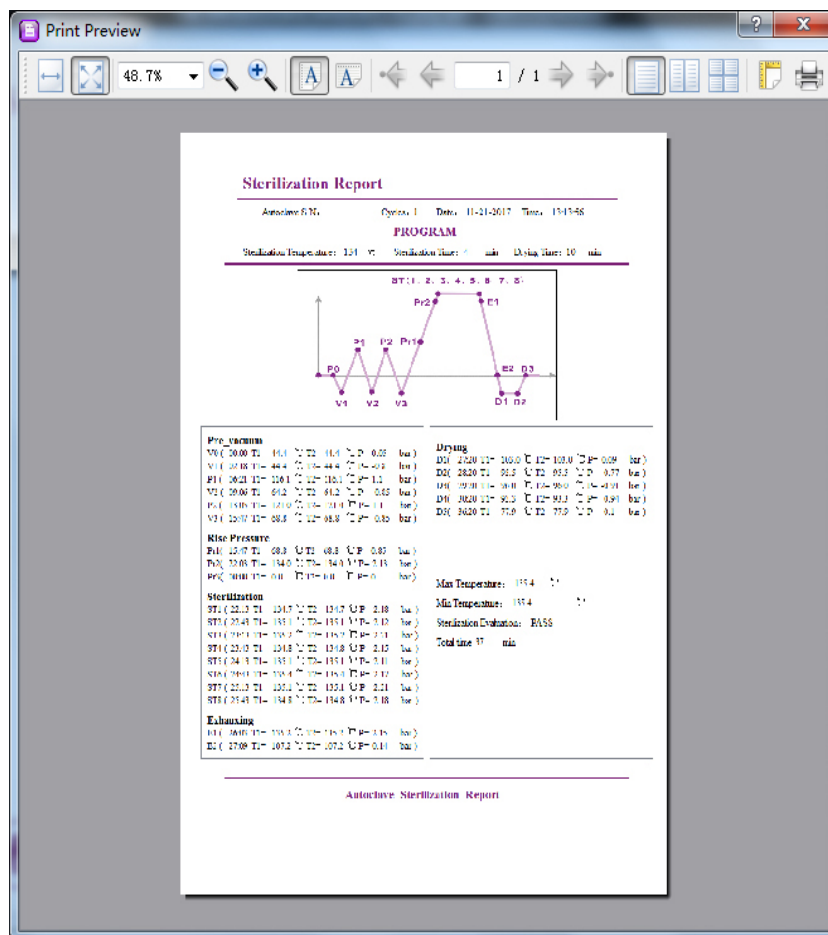

11. Press this button, to delete the chosen record.

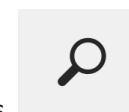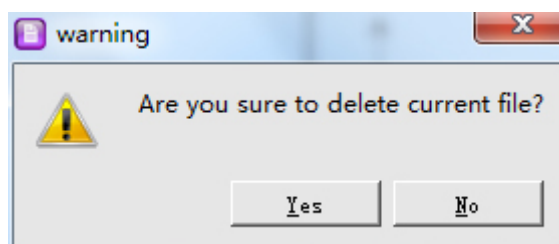12. Press this button, so that users can search the record.

13. The hidden buttons (they appear when the mouse is close) are: the left and right scroll bars in the right screen and the three icons in the upper right corner of the software: minimized, maximized, and closed.

14. The record is shown in the right screen.